



## “AccuFit” MRI Headrest Drape Instructions For Model # BS-1003

The “AccuFit” MRI Headrest drape is designed to protect your equipment and patients from any oils, make-up or contact skin issues. This headrest cover will fit the following headrests:



**Headrest #1**



**Headrest #2**



**Headrest #3**



**Headrest #4**

1. Open with the elastic ends matching the form of the headrest if you have headrest #1. Start with the top and wrap around the ends. **FIG. 1**



**Headrest #1 FIG.1**

2. Pull the drape around the entire headrest as shown in **FIG. 2**



**FIG. 2**

3. For headrest #2, #3 and #4 make sure the elastic is down and the flat portion on top. Start the drape at the horseshoe end. (**FIG. 3**) Keep it level and pull it across the top around to the other end.



**FIG.3**

4. Adjust as necessary. Make sure you have complete visualization of the mirror for the patient.

