



## AccuFit™ Fischer/Siemens® MammoTest® Table Drape Instructions Model #BS-1500 (Patent Pending)

“AccuFit” Stereotactic Table Cover is designed to protect the patient and your MammoTest® Stereotactic Breast Biopsy System.

### 1. English- en

Unfold and begin placement by starting at the head of the table.  
Align the aperture hole in the table with the drape. Slip the elastic  
over the foot of the table. Adjust as needed.

### Français – French - fr

Dedoublez et commencez le placement en commençant par la tête de  
la table. Alignez le trou d'ouverture dans la table avec le drap. Glissez  
l'élastique au-dessus du pied de la table. Ajustez selon soit nécessaire.

### Deutsch – German - de

Breiten Sie aus und fangen Sie Platzierung an, indem Sie am Kopf der  
Tabelle beginnen. Richten Sie die Blendenöffnung Bohrung in der  
Tabelle mit drapieren aus. Gleiten Sie das Gummiband über dem Fuß  
der Tabelle. Justieren Sie, wie gebraucht.

### Italiano – Italian - it

Spiegate e cominciate la sistemazione cominciando dalla testa del  
tavolino. Allineate il buco dell'apertura nel tavolino con il drappo o  
tessuto. Infilate l'elastico sopra il piede del tavolino. Aggiustate  
secondo sia necessario.

### Español – Spanish - es

Desdoble e inicie la colocación empezando en la orilla superior de la  
mesita. Alinee la abertura del agujero en la mesita con el paño.  
Deslice el elástico sobre el pie (la pata) de la mesita. Ajuste según sea  
necesario.

