# AccuFit™ Fischer/Siemens MammoTest® Autoguide Drape Instructions Model# BS-1501 (Patent pending)

“AccuFit” Stereotactic Autoguide Drape is designed to protect you and your MammoTest® Stereotactic Breast Biopsy System from infectious body fluids generated during a stereotactic breast biopsy.

## 1. English- en

Unfold and begin placement by starting at the camera. The camera end has the long white strip across the plastic. Remove white sticky and place along the back of the camera.

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## 2. English- en

Find the “Place Paddle Here” sticker. On each side of the sticker are two slits that fit the frame of the paddle holder. Push the slits to the bottom of the paddle holder.

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**1. English- en**

Place the plastic paddle in the small clear fluid retention pouch and fold plastic over paddle frame.

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**2. English- en**

Where the drape splits pull the stretchy blue material tight around the Autoguide boot. Peel the stick tab on the absorbent end and cross over the other absorbent to make a tight fit around the Autoguide boot.

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**1. English- en**

Remove sticky tabs on both sides of the camera plastic. Pull the plastic tight over the camera to eliminate any wrinkles.

**The unit shown with proper placement of the draping system.**

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