

Operating Instructions

Operating in the MR Environment

When operating the 7500FO in the MR (magnetic resonance) environment, observe the following safety considerations:



Use only Nonin-branded 8000FC or 8000FI Fiber Optic sensors. Do not use cables or sensors that contain conductive wires.



The 7500FO and the connectors for the fiber optic sensors contain ferrous material and must be kept outside the MR room.

WARNING: To avoid injury or potential equipment damage, always keep the oximeter and metal end of fiber optic cable outside of the MR room in the control/observation area

MR Conditions

MR OPERATIONS:
USE ONLY NONIN Fiber
Optic Sensors
(Sensors containing electrical
conductors will cause patient burns)



WARNING: The battery charger cannot be used in the MR environment

When operating the 7500FO in the MR environment, observe the following installation recommendations

- Install the 7500FO outside the MR room. The monitor can be firmly attached to a fixed object using the mounting hole (1/4 -20 thread) on the bottom of the device.
- If interference is suspected to the MR image or to the 7500FO, contact Technical service