

MRI-SAFE CONRAD® THERMAL BLANKET INSTRUCTIONS

(Always have PATIENT SIDE facing the patient)

1

ANESTHESIA

Anesthetize on an active warming pad (HoverHeat) and cover with the MRI Safe – ConRad Thermal Blanket.



2

Moving Patient to the MRI Scanner


Circumferentially wrap the patient with a fleece blanket and the MRI Safe – ConRad Thermal Blanket to cover the patient as completely as possible.



3

In the MRI Scanner

Place a second MRI Safe – ConRad Thermal Blanket on the MRI scanner table PATIENT SIDE up and place the wrapped patient on the MRI scanner table.

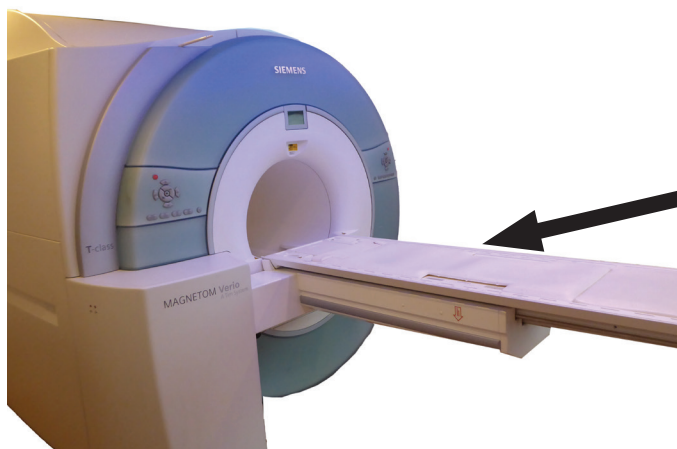

Epik Medical Innovations

3871 Concomly Rd S • Salem, OR 97306 • Tel# 503-931-2932

www.EpikMedical.com