

MRI PATIENT CAMERA SYSTEM SET UP

Easy as 1-2-3 !

1



Mount camera's on a wall facing both ends of MRI tube.

WARNING: Keep camera's at least 6 FT. away from MRI Machine. MRI SAFE Tools are only to be used inside the MR room to mount camera's

2



Run the special MRI power cable from both camera's to nearest AC outlet and plug in. Some MRI rooms are already equipped with special AC

3



Connect DC Power supply to monitor tablet. Plug the DC adaptor into a wall outlet. Raise up the left/right antennas located on top of tablet. Slide the ON/OFF switch to ON to power up. You should see video of one camera. Tap the screen to see both camera's. You can select a specific camera by tapping on the video screen of that camera then tap again. The screen should now display a single view.

WARNING: ATTENTION: disclaimer Do not bring monitor tablet into MRI scan room or near the magnet. This could damage the device. All MRI power cables should be run above the ceiling tiles away from any electrical/MRI cables. MRI power cables should not be run along the floor. You must use nonmagnetic MRI certified tools for installation. The MRI Video camera system has been tested to operate in 1.5T & 3T. We are not responsible for site installation, providing an AC power outlet to power the system, or any other additional costs to install the video camera system. You have been notified about the precautions and we are not liable for any damages.